



Spherical Aberration Annotation

1. Ensure the Right pane is visible.
2. Go into Arrange attributes.
3. Select W(0,4) Spherical Aberration and move to the right Selected window with ►

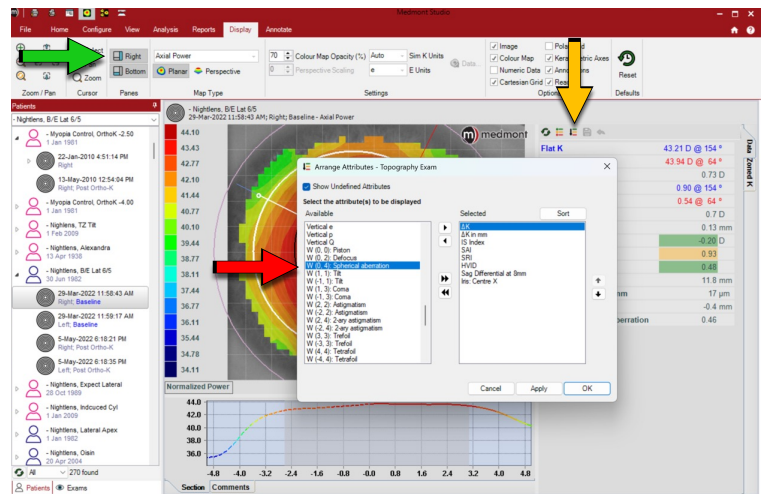


4. Change the readout to um by going in to the Edit Attribute. Select W(0,4) Spherical aberration and Edit

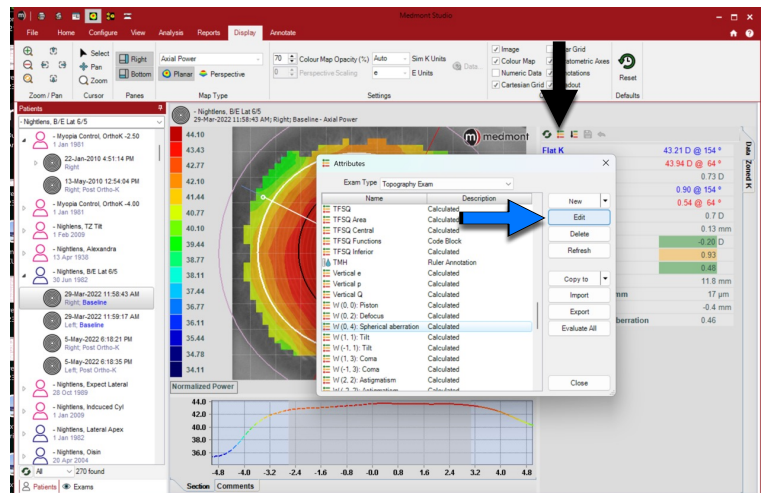

5. Uncheck the Scientific Format.

For good myopia control in orthok, you want a positive increase of 0.8um in spherical aberration. So +0.8um over the baseline value.

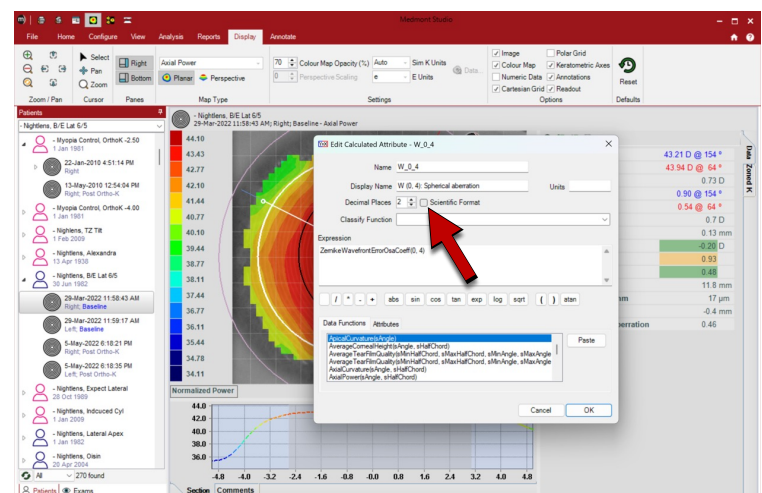



ORTHO-K NIGHT LENS
overnight vision correction