

SYMPTOMS

Lens drying, discomfort. Unstable, reduced VA.

SIGNS

Protein coating on lens surface (top) or in lens carrier at edge (bottom). Poorer wetting.

Eye may be red, conjunctiva may develop follicles or papillae.

CAUSE

Poor cleaning, or patients can be prone even with diligent cleaning with MPS.

ACTION

Daily protein cleaning regime with Avizor EverClean / Rubbing with surfactant such as Boston Cleaner. Or protein removing tablets 3x weekly.

A new lens may be required. More frequent lens replacement.

