



OCULUS COMPARISON MAPS GUIDE

For clinical support or advice send comparison maps of both Tangential and Refractive, as below.


To set up the standard colour and scale to view maps, adjust in *Settings* as shown:

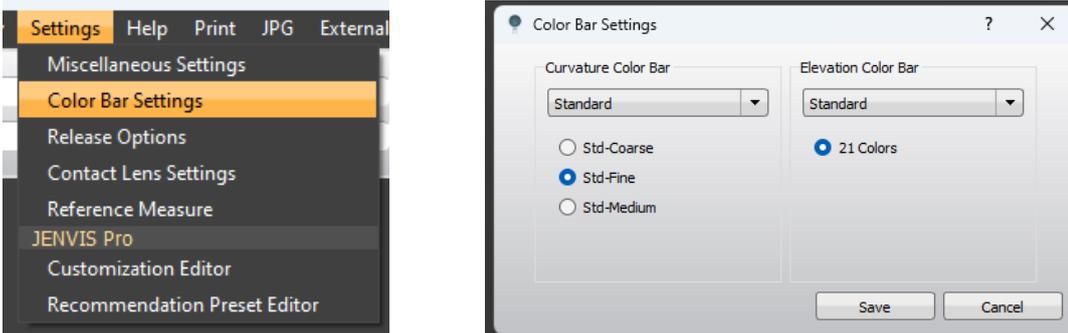


Select 'Display' > 'Compare 2 Exams':

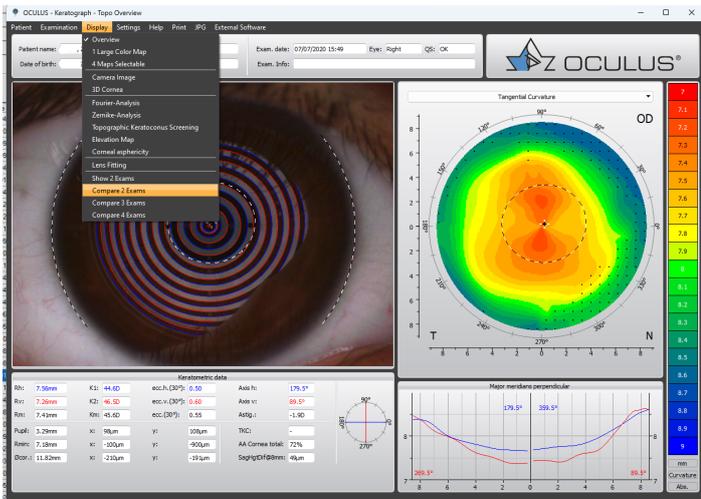


Top left baseline, bottom left newest. Select 'Tangential Curvature' in 'Difference B-A:' Dropdown:

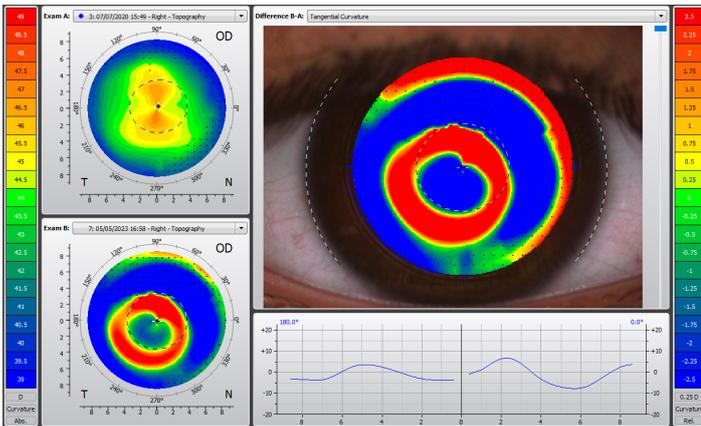

*Note: Ensure most appropriate scale is selected:

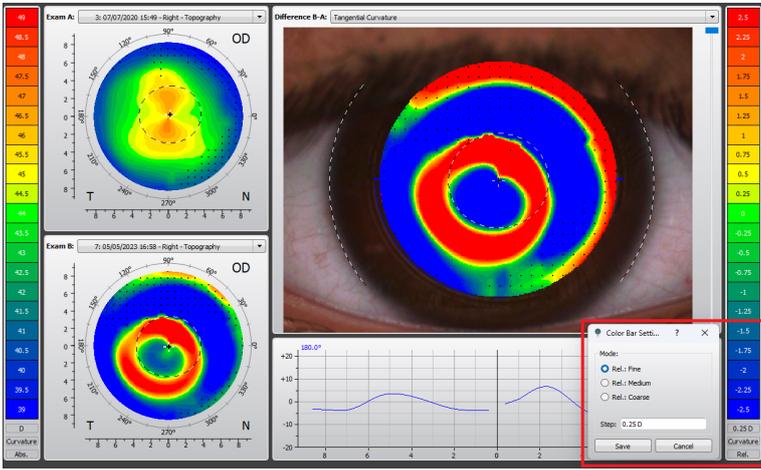

Select 'Refractive' in 'Difference B-A:' Dropdown:

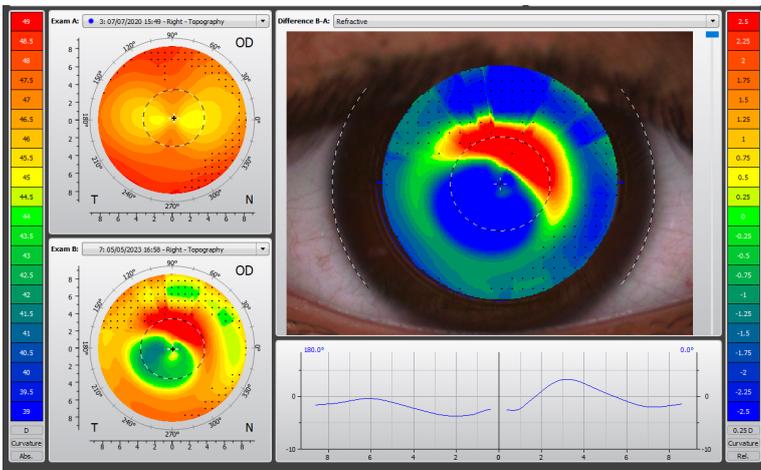

Upload to Clinical Support Form on scotlens.com or email to support@scotlens.com